

# SUPERNO<sub>2</sub>VA® ET MASK AND SYSTEM

## HYPOXEMIA FROM UPPER AIRWAY OBSTRUCTION?



To ensure an air-tight seal, apply firm downward pressure to the mask and use the headstrap to secure in place.

Is your SuperNO<sub>2</sub>VA® Et mask sealed properly on the patient's face? If the reservoir bag doesn't pressurize:

- Ensure SuperNO<sub>2</sub>VA Et mask seal is air-tight.
- Address any leaks from the mouth.
- Check for leaks and close any open ports.



Set FGF to 10 LPM



Adjust APL Valve to 10 cm H<sub>2</sub>O




Connect the sampling line to the gas sampling port

## TROUBLESHOOTING: ADDRESSING ORAL LEAK



Manual mouth closure

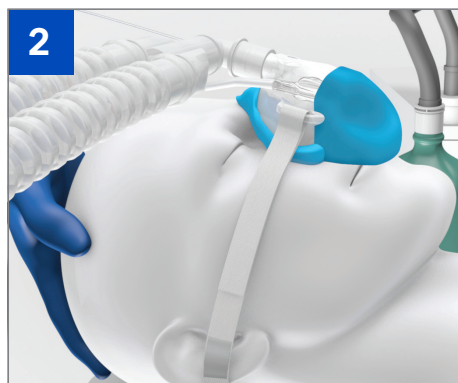

Slight head flexion



Sub-mental pressure on the tongue